

## Ling Sound Test

The Ling sound test is used to check that an individual has access to sounds across the full speech spectrum.

Detection: The individual indicates a sound is heard

Identification: The individual repeats the sound or points to a card

### Directions

1. Say each sound, one at a time, and in no particular order.  
Use a normal speaking volume. Remove all lipreading and visual cues.
2. Repeat this test with each processor (or hearing aid) alone.
3. Check distance hearing by delivering the sounds from 2-3 meters away.
4. Document the results.
5. Discuss any difficulties with your hearing professional.



## Keep track of your responses

Date	ah	m	oo	ee	sh	ss

✓ = Correct response

- = No response

() = If the wrong sound is repeated back, the person administering the test should write the sound that you make (e.g. if the sound presented is "sh" and you say "s" your partner should write "s" in the box.

MED<sup>9</sup>EL

ah



MED<sup>9</sup>EL

m



MED<sup>9</sup>EL

oo



MED<sup>9</sup>EL

ee



MED<sup>9</sup>EL

sh



MED<sup>9</sup>EL

ss

